

ERGOCONCEPT CATALOGUE

Revolutionary mobility solutions ! 2020



Version 1.3 - September 2020

ACKNOWLEDGEMENT



The ErgoConcept adventure began six years ago !

So many paths have been travelled and so many enriching encounters since our launch.

Today, a united and smiling team of a dozen of people gives the best of itself to bring adapted solutions and provide all the services to give satisfaction to our partners.

We warmly thanks all our providers, ergotherapists, health professionals, and of course, all wheelchairs and scooter's users for their trust. This motivates us every day to develop new revolutionary mobility solutions.

We continue to surpass ourselves individually and collectively to ensure that this great story will continue hand in hand with our loyal partners. Our constant concern is to improve the daily lives of people with reduced mobility.

ErgoConcept TEAM



Signature of a partnership contract with TomZorg, expert in medical equipment in the Netherlands.



A growing experts network...

After consolidating a network of more than 850 experts resellers in France, and in response to growing demand abroad, we officially launched ErgoConcept for export in 2020 !

We are thus very proud to establish strong partnerships with highly qualified specialists in sales and consulting in medical equipment. This is an additional development opportunity for the company which has been growing continuously since its creation in 2014.

As a part of our development and in order to constantly improve our quality of service, we make strong commitments to our partners.

ErgoConcept is committed to:

- Be always available and listening to our partners, by email, phone and visit in order to provide clear and precise answers to any remarks as soon as possible.
- To keep our commitments in the absolute respect of the given word and to fully assume our responsibilities.
- To put innovation at the heart of our concerns to ensure technical development, but also at the level of the services provided in order to constantly deliver new solutions.
- To find inspiration from users and health professionals (resellers, ergotherapists, rehabilitation doctors) in order to remain a driving force in our market sector.
- Perpetuate our spirit of initiative in order to mark the difference with our colleagues, continue to inspire the market without ever becoming a "follower" or indulging in the too easy but ineffective practice of copying.
- Always rely on our network of partners and refrain absolutely from any form of direct sale or sale by any vectors that seem inappropriate to us.
- Refrain from any sale when our solutions seem unsuitable in the medium term.
- To continue to hold our delivery deadlines and to provide fast and efficient after-sales service solutions.
- To consider 'Customer Satisfaction' as a daily driving force.
- Always keep in mind that at the end of the process, there are fragile people who need our full attention and professionalism.

Professionally yours,
For the whole team, Martin Vernay / ErgoConcept President



Electric wheelchair

- Ergo 07L	5
- Ergo 08L	9
- Ergo 09L	13

Scooter & BiKube

- Ergo SL	17
- BiKube	18

Accessories

- Confort	19
- Pilotage	30
- Transport	35
- Pratique	39

Parameterization control	42
--------------------------------	----

Loan & After-sales services	45
-----------------------------------	----

A day in E08L wheelchair	46
--------------------------------	----

ERGO 07L CLASSIC



LIGHT
FOLDABLE
COMPACT

An economical and functional solution

Color	Autonomy Battery	Overall wide	Seat size (L x l x H)	Folded size (L x l x H)	Max speed
  	12 km	59 cm	44x40x5 cm	70x59x29 cm	6 km/h
Turning Radius	Height Floor/seat	Overall length	Obstacle clearance	Safety Sloap	Max weight
75cm	44 cm	80 cm	5 cm	10°	100 Kg

- . Ultra light: **22 kg**
- . Foldable: **5 seconds**
- . Adapted to **small car trunk/plane**
- . Usable **Indoors/Outdoors**
- . Front/Back wheel: **8/7,5 inches**
- . Power: **2 x 150 W**

ERGO 07L VOYAGER

PACK INCLUDE :

Additional
Battery
(Autonomy: 24 km)

+

Cable Y

+

Anti-puncture
Bomb



The perfect model for frequent travelers

Color	Autonomy Battery	Overall wide	Seat size (L x l x H)	Folded size (L x l x H)	Max speed
	12 km	59 cm	44x40x5 cm	70x59x29 cm	6 km/h
Turning Radius	Height Floor/seat	Overall length	Obstacle clearance	Safety Sloap	Max weight
75cm	44 cm	80 cm	5 cm	10°	100 Kg

. Ultra light : **23 kg**

. Front/Back wheel : **8/7,5 inches**

. Adapted to **small car trunk/plane**

. Power: **2 x 150 W**

ERGO 07L PRESTIGE



PACK INCLUDE:

Additional Battery
(Autonomy 24 km)

+

Comfort Seat & Back

+

GEL Armrest

+

Carry Bag

+

Cable Y

+

Anti-puncture
Bomb

+

Batterie Charger
Adapter

A high quality finition for more comfort !

Color	Autonomy Battery	Overall wide	Seat size (L x l x H)	Folded size (L x l x H)	Max speed
  	12 km	59 cm	44x40x5 cm	70x59x29 cm	6 km/h
Turning Radius	Height Floor/seat	Overall lenght	Obstacle clearance	Safety Sloap	Max weight
75cm	44 cm	80 cm	5 cm	10°	100 Kg

. Ultra light : 23 kg

. Front/Back wheel : 8/7,5 inches

. Adapted to small car trunk/plane

. Power: 2 x 150 W

Find the perfect solution for your needs:

Range	E07L		
Model			
Visual			
Gross Weight	22Kg	23Kg	23Kg
Naked Weight(1)	19Kg		
Indoors Use	***	***	***
Outdoors Use(2)	**	***	***
Overall width	59 cm	59 cm	59 cm
Front wheel	7,5 inches		
Back wheel	8 inches		
Folding time	5 seconds		
Autonomy	12 Km	24 Km	24 Km
Power	2 x 150W		
Max weight	100Kg		
Model +	A compact solution, customizable with several accessories		

(1) Without battery & Textile

(2) In a suitable environment

CLASSIC



Weight
20,5 kg

+ battery &
textile: 4,5 kg

MULTIPLE-SETTINGS

ULTRA LIGHT

DOUBLE-PALLET

RECLINING BACK

Innovative, compact and equipped with many customizable settings (backrest, footrest, armrest, joystick...), ERGO 08L is an ultra POLYVALENT wheelchair ! With its new simplified folding system (3 sec) and its double-pallet footrest, it opens up new perspective to users with reduce mobility.

NEW Model: a concentrate of innovations !

Color	Battery Autonomy	Overall wide	Width btw armrests	Folded size (L x l x H)	Max speed
	30 km	60 cm	45-49 cm	72x60x36 cm	6 km/h
Turning radius	Height Floor/seat	Overall length	Power	Safety sloap	Max weight
72 cm	52 cm	80/105 cm	2 x 200 W	6°	115 Kg

- . Ultra light: **20,5 kg** (+ battery 4,5 kg)
- . Foldable: **3 seconds**
- . Adapted to **small car trunk/plane**

- . Front/Back wheel : **8/10 inches**
- . Reclining back: **3 positions**
- . Footrest **double-pallet**



Ergo 08L VOYAGER

PACK INCLUDE

Carry Bag
+
Armrest Bag
+
Loading sheet
+
Anti-puncture Bomb

The perfect solution for frequent travelers !

Color	Battery Autonomy	Overall wide	Width btw armrests	Folded size (L x l x H)	Max speed
	30 km	60 cm	45-49 cm	72x60x36 cm	6 km/h
Turning radius	Height Floor/seat	Overall length	Power	Safety sloap	Max weight
72 cm	52 cm	80/105 cm	2 x 200 W	6°	115 Kg

- Ultra light: **20,5 kg** (+ battery 4,5 kg)
- Foldable: **3 seconds**
- Adapted to **small car trunk/plane**
- Front/Back wheel : **8/10 inches**
- Reclining back: **3 positions**
- Footrest **double-pallet**

PRESTIGE

PACK INCLUDE

Carry Bag
+
Armrest Bag
+
Loading sheet
+
Anti-puncture Bomb
+
Smartphone Support
+
USB Charger
+
Chargeur Adapter



The model that will revolutionized your daily life !

Color	Battery Autonomy	Overall wide	Width btw armrests	Folded size (L x l x H)	Max speed
	30 km	60 cm	45-49 cm	72x60x36 cm	6 km/h
Turning radius	Height Floor/seat	Overall length	Power	Safety sloap	Max weight
72 cm	52 cm	80/105 cm	2 x 200 W	6°	115 Kg

- . Ultra light: **20,5 kg** (+ battery 4,5 kg)
- . Foldable: **3 seconds**
- . Adapted to **small car trunk/plane**
- . Front/Back wheel : **8/10 inches**
- . Reclining back: **3 positions**
- . Footrest **double-pallet**

ERGO

08 L

GAMME ERGO 08L advantages :



Reclining
Back
3 positions



Adjustable
Backrest
in height



Foldable
in one gesture
en 3 secondes



Adjustable
Footrest
double-pallet



Adjustable
Joystick
4 positions



Comfort
Seat
dual-density



Back
Wheel
10 inches



Double
Battery
Autonomy: 30km



Aluminium
Fork
Blue design



CLASSIC



Max weight

MULTI-POSITIONS
3 MODELS/3 SIZES
RECLINING BACKREST
CLIP'GO SYSTEM

Ergo 09L is an ultra comfortable folding electric wheelchair with many adjustments (joystick, armrest, footrest...). With its reclining backrest (5 positions) and the exclusive CLIP'GO system (possibility to lighten the wheelchair by 11 kg), it is an ideal mobility solution for daily rides as for travelling. Available in 3 sizes.

The very practical multi-positions wheelchair

Choose your size	COMPACT	ORIGINAL	XL
Overall width	61 cm	66 cm	67 cm
Width btw armrest	43 cm	48 cm	53 cm

Color	Battery Autonomy	Overall width	Seat size (L x l x H)	Folded size (L x l x H)	Max speed
	26 km	*C: 61 cm O: 66 cm XL: 67 cm	C: 44 x 42 x 6 cm O: 44 x 45 x 6 cm XL: 44 x 45 x 6 cm	C: 80 x 61 x 35 cm O: 80 x 66 x 35 cm XL: 80 x 67 x 35 cm	6 km/h
Turning radius	Height Floor/seat	Overall length	Obstacle clearance	Safety sloap	Max weight
80 cm	51 cm	102 cm	6 cm	8°	150 Kg

*C: Model Compact - O: Model Original - XL: Model XL

VOYAGER

PACK INCLUDE

Carry Bag

+

Armrest Bag

+

Plane Battery



The ideal wheelchair to travel in total freedom !

Choose your size	COMPACT	ORIGINAL	XL
Overall width	61 cm	66 cm	67 cm
Width btw armrest	43 cm	48 cm	53 cm

Color	Battery Autonomy	Overall width	Seat size (L x l x H)	Folded size (L x l x H)	Max speed
	26 km	*C: 61 cm O: 66 cm XL: 67 cm	C: 44 x 42 x 6 cm O: 44 x 45 x 6 cm XL: 44 x 45 x 6 cm	C: 80 x 61 x 35 cm O: 80 x 66 x 35 cm XL: 80 x 67 x 35 cm	6 km/h
Turning radius	Height Floor/seat	Overall length	Obstacle clearance	Safety sloap	Max weight
80 cm	51 cm	102 cm	6 cm	8°	150 Kg

*C: Model Compact - O: Model Original - XL: Model XL

PRESTIGE

PACK INCLUDE

Gel Armrest
+
Carry Bag
+
Armrest Bag
+
Loading sheet
+
USB Charger
+
Anti-puncture bomb



The model that revolutionized your daily life !

Choose your size	COMPACT	ORIGINAL	XL
Overall width	61 cm	66 cm	67 cm
Width btw armrest	43 cm	48 cm	53 cm

Color	Battery Autonomy	Overall width	Seat size (L x l x H)	Folded size (L x l x H)	Max speed
	26 km	*C: 61 cm O: 66 cm XL: 67 cm	C: 44 x 42 x 6 cm O: 44 x 45 x 6 cm XL: 44 x 45 x 6 cm	C: 80 x 61 x 35 cm O: 80 x 66 x 35 cm XL: 80 x 67 x 35 cm	6 km/h
Turning radius	Height Floor/seat	Overall length	Obstacle clearance	Safety sloap	Max weight
80 cm	51 cm	102 cm	6 cm	8°	150 Kg

*C: Model Compact - O: Model Original - XL: Model XL

Experience the new system **CLIP&GO**

EXCLUSIVE ! With Clip&Go, lighten your wheelchair by removing the back wheels in less than one minute.

It's very easy, just check the video presentation on our website.

Tips: It is also possible to remove the battery (4,5 kg), the wheelchair then weights only 17 kg !



Adjust the height of your armrests !



For more comfort, it is now possible to adapt the height of E09 armrests.

You can choose from the following levels:

- 17 cm
- 19 cm
- 21 cm

ERGO

SL

15 km 



Ultra light: 24 kg
(+ battery 2 kg)



AUTOMATIC
FOLDING



Folded dimensions:
26 x 64 x 45 cm

The 4-wheel scooter that folds with a remote control !

Color	Battery Autonomy	Overall width	Overall length	Power	Max speed
	15 km	45 cm	94 cm	120 W	6 km/h
Turning radius	Height Floor/seat	Max weight	Wheel Front/Back	Safety Slope	Voltage
1,55 m	52 cm	125 kg	6/7 inches	12°	24 V

Optional accessories:

- Carry Bag
- Charge station
- Loading sheet
- Armrest bag





The NEW hands-free rollator !

Weight	Lenght	Width	Max weight	Folded dimensions
8,62 kg	73,5 cm	62 cm	115 kg	98 x 62 x 22 cm
Seat height	Wheel diameter	Front frame height	Inseam height	Handle height
65 cm min.	10 cm	79,5 cm	65 à 90 cm	85 à 110,5 cm

BiKube is the new home mobility device that combines innovative features with elegant design.

Created for an indoors use, it helps people with reduce mobility to maintain their stabilty (sitting or standing)and independance in daily life. The perfect solution to regain total self-confidence in usual activities. The BiKube is also widely use for rehabilitation whether at home or in a medical center.

ACCESSORIES / COMFORT



Include in
standard
model

E07L



SEAT & BACKREST STANDARD **ERGO** 07 L

Ergonomic and practical seat and backrest provided with Ergo 07L wheelchair. Velcro fixation.

HIGHLIGHTENED SEAT **ERGO** **ERGO** 07 L 08 L

This elevated seat allows to adjust up to 8cm the seat height.



Include in
standard
model

E08L



SEAT & BACKREST COMFORT **ERGO** **ERGO** 07 L 08 L

Comfortable seat and backrest. Moulded bi-density memory foam.

Name	Reference	Dimensions (LxIxH)	Compatibility
Standard Seat	7T030000	38x42x5 cm	Ergo 07/08
Standard Backrest	7T040000	38x42 cm	Ergo 07/08
Elevated Seat	7T020000	38x42x8 cm	Ergo 07/08
Pack Comfort	7T160000	38x42x5/8 cm (seat)	Ergo 07/08

SEAT & BACKREST STANDARD

ERGO
09 L

Ultra-comfortable memory foam seat, included in standard E09 model.



DEPTH PACK

ERGO
09 L

This pack allows to add an optional seat extension of 5 or 7 cm in length.

Name	Reference	Dimensions (LxIxH)	Compatibility
Standard Seat	9T040000	44X43X8/6/9 cm	Ergo 09
Standard Backrest	-	44x42 cm 47x45 cm	Ergo 09 (C*) Ergo 09 (O & XL*)
Pack 5 cm Depth	NRT01	49X44X8 cm	Ergo 09
Pack 7 cm Depth	NRT02	51X44X8 cm	Ergo 09

*C: Compact Model - O: Original Model - XL: XL Model

MODULARE SEATING SYSTEM (MSS)



NEW !

The new MSS backrest has new features for comfort and positioning.

The back tension is adjustable via the 4 internal horizontal straps which can be individually tensioned for personalised and precise adjustment (Fig 1).

The backrest curvature adjustment system allows to adjust the shape and the sensation to suits individual requirements. Possibility of adjusting the right or left curves to resolve symmetrical and asymmetrical posture issues (Fig 2).

The fastening system also offers a multitude of adjustments in height, depth and inclination.

The MSS cushion combines comfort and stability. It is composed of 3 zones:

- Stability base with ischiatic HR foam tray
- Moulding of visco-elastic foam
- Finishing foam with popliteal cavity and clearance

This structure facilitates a balanced and regular immersion by limiting forward slipping, and ensure a good level eschar prevention. Soft and comfortable cushion.



Fig 1

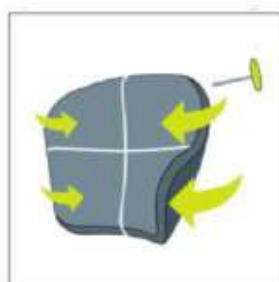


Fig 2

Name	Reference	Dimensions (LxlxH)	Compatibility
AME Backrest include	7T10L000	Seat: 40x45 cm Backrest width: 40 cm	Ergo 07/08/09 C*
AME Backrest include	9T100000	Seat: 45x45 cm Backrest width: 45 cm	Ergo 09 O* et XL*

*C: Compact Model - O: Original Model - XL: XL Model

LEG REST



*New model. Choose from the table below the side that suits you to hold the right (R) or left (L) side.

The leg rest, ajustable in lenght (4 positions from 47 to 57 cm), allows an optimal support of the leg and a wide amplitude of inclination. The adjustable calf support enhance user comfort.



LEG SUPPORT BELT



This accessory provides support for the thighs or calves to prevent a lower limb from slipping or spreading appart.



Name	Reference	Compatibility
Leg rest	7M07000D (R) 7M07000G (L)	Ergo 07/08
Leg rest	9M07000D (R) 9M07000G (L)	Ergo 09
Support belt	7T150000	Ergo 07/08/09

ACCESSORIES / COMFORT

FOOTREST STANDARD **ERGO** 07 L



Model supplied with the Standard wheelchair
(Black or silver)



FOOTREST LARGE (L) **ERGO** 07 L



5 cm extra depth / anti-slip



Name	Reference	Compatibility
Footrest Standard (black)	7M06B000	Ergo 07
Footrest Standard (silver)	7M060000	Ergo 07
Footrest L	7M070000	Ergo 07

ACCESSORIES / COMFORT

DOUBLE-PALLET ✓ FOOTREST STANDARD ERGO 08 L

Model supplied with the Standard wheelchair, the double-pallet footrest makes transfers easier.

Practical and easily removable, this footrest is adjustable in height and depth.



FOOTREST MONOBLOC ERGO 08 L



Optional

NEW !

Among new accessories available, it is now possible to purchase a monobloc footrest on the E08L model.

Optional accessory.

Name	Reference	Compatibility
Footrest Double-pallet	-	Ergo 08 (included)
Footrest Monobloc	8M060000	Ergo 08

FOOTREST STANDARD ERGO 09 L



Footrest supplied with the wheelchair E09L, ergonomic and comfortable.

FOOTREST ADJUSTMENT SYSTEM ERGO 09 L



The E09L footrest is adjustable in height and length with these extenders (included).

Do not fix several extenders in length.

Name	Reference	Compatibility
Footrest	9M030000	Ergo 09



FOOTREST EXTENSIONS ERGO 09 L

NEW !

In order to improve the footrest comfort, it is possible to add an extension of 5 cm in width or depth.



EXTENSION IN DEPTH



EXTENSION IN WIDTH



Name	Reference	Compatibility
Extension	9M03L000	Ergo 09

ARMREST STANDARD



Included with the standard wheelchair

ARMREST GEL



Comfortable, depth-adjustable armrest with gel foam.

ARMREST GUTTER



Armrest for an optimum forearm support



Name	Reference	Dimensions (LxI)	Compatibility
Armrest Standard x1	7M020000	32,5x5 cm	Ergo 07/08
Armrest Standard x1	9M020000	31X7 cm	Ergo 09
Gel x2	7M01G000	30,5x6 cm	Ergo 07/08
Gel x2	9M01G000	26x10 cm	Ergo 09
Gutter x1	7M220000	42x17x24 cm	Ergo 07/08/09

HEAD SUPPORT



CUSHION



FIXATION

This accessory is adjustable in height and inclination to improve head rest.

Optimized fastening system (easy to install/adjust)

Name	Reference	Compatibility
Head support	7M030000	Ergo 07/08/09

PANTOGRAPH



This accessory allows the joystick to be shifted out from the armrest (compatible right or left side).



* Note that the use of the pantograph increases the overall width by 5 cm.

CANE HOLDER



Ideal for attaching a cane easily to the back of your wheelchair. Possibility to install 2 cane holders.



Name	Reference	Compatibility
Pantograph	7M050000	Ergo 07/08/09
Cane holder	7P010000	Ergo 07/08/09

THIRD-PARTY CONTROL



3-WIRE
SWITCH

BACK JOYSTICK
FIXATION*

Fixed on the wheelchair backrest, the third-party control allows the accompanying person to take control of the wheelchair.

The switch makes it easy to choose whether to activate the user joystick or the third-party control.

PARAMETERIZATION COMMAND

This accessory, reserved for professionals, is the essential tool for setting the wheelchair according to the need of each user (acceleration, speed...)

Parameter setting table at the end of the catalogue.



*Possibility to acquire the back joystick fixation independently

Name	Reference	Compatibility
Third-party control	7E100000	Ergo 07/08/09
Fixation Joystick	7M170000	Ergo 07/08/09
Parameterization command	9E090000	Ergo 07/08/09

FRONT WHEEL PNEUMATICS



Choosing the right wheel to your daily use is essential for a proper use of the wheelchair:

. **Pneumatics wheel**

Mixte use (indoors/outdoors), offers comfort in all circumstances. Obstacle clearance: 5/6 cm.

FRONT WHEEL BANDAGE



. **Bandage wheel**

Mainly for indoors use. Requires no maintenance.

FRONT WHEEL BANDAGE SLIM



. **Bandage Slim wheel**

For indoors use only. Optimised handling.

All models are sold in pairs:

Name	Reference	Compatibility
Pneumatics wheel	7M080000	Ergo 07/08/09
Bandage wheel	7M090000	Ergo 07/08/09
Bandage Slim wheel	7M100000	Ergo 07/08/09

ANTI-PUNCTURE BOMB



This accessory allows to re-inflate a wheelchair tire in case of a flat tire, for a distance of about approximatively 3 km, before proceeding with necessary repairs.



Reference	Compatibility
7M230000	Ergo 07/08/09

TRACKBALL



The steering ball enables an optimized grip of the joystick for more manoeuvrability when driving the wheelchair.

Reference	Compatibility
7M240000	Ergo 07/08/09

BATTERY 09L

ERGO
09 L



4,45 Kg

PLANE BATTERY

ERGO
09 L



2,9 Kg

Conform to the standards of the main airline and shipping companies

BATTERY

ERGO ERGO
07 L 08 L



Conform with all the main airline and shipping companies

CHARGER ADAPTER

ERGO ERGO
07 L 08 L



Allows to charge a battery independantly of the wheelchair

Name	Reference	Compatibility	Voltage	Amperage	Autonomy
Battery 09	9E030000	Ergo 09	24 V	15 A	25 km
Plane Batt.	9E040000	Ergo 09	24 V	10 A	18 km
Battery 07	7E030000	Ergo 07/08	24 V	8 A	12 km
Adapter	7E010000	Ergo 07/08	-	-	-



Included in
standard
model
E_{08L}



CABLE Y DOUBLE BATTERY

ERGO_{07 L} ERGO_{08 L}

This cable connects the 2 wheelchair batteries and allows to use them while charging simultaneously. The essential accessory to double the autonomy up to 24 km.

TIPS : The black end connects to the power modulator, and the either two to the battery on either side of the wheelchair.

Reference	Compatibility
7E050000	Ergo 07/08

ERGO_{SL}



CHARGE STATION

This accessory allows to charge the 2 batteries (2 x 1 kg) independently of the Ergo SL scooter. Remove the batteries and simply place them on the charging station without taking up space in home.

Reference	Compatibility
SLE04	Ergo SL

ERGOLIFT



Electric charging accessory for effortless transfer of the wheelchair into a car trunk.

Demonstration video on our website:
www.ergoconcept.net

ERGOLIFT PACK

For any use without an ErgoConcept product, there is the pack:



The Ergolift is delivered without battery and can be connected directly to ErgoConcept products.

Dimensions open: 1780x85x70 cm
Dimension folded: 70x40x23 cm
Max charge: 30 Kg

Name	Reference	Height/weight	Compatibility
Ergolift	7E210000	1,50m/13,5Kg	Ergo 07/08/09/SL
Pack Ergolift	7E240000	-	-

CARRY BAG

E_{07L}E_{08L}E_{09L}

The ideal accessory to move the wheelchair or scooter easily.




CARRY BAG E_{SL}



Name	Reference	Compatibility
Carry Bag	7T050000	Ergo 07/08
Carry Bag	9T050000	Ergo 09
Carry Bag	SLT050000	Ergo SL

LOADING SHEET

The loading sheet is the essential accessory to easily carry your mobility solution in the trunk of the car.

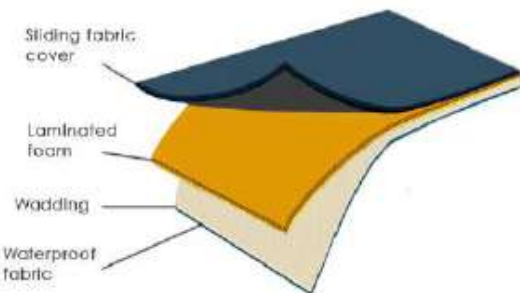

Demonstration on our website:
www.ergoconcept.net/videos

Designed with laminated foam, it protects your equipment and facilitates transfer thanks to its sliding fabric cover.



Name	Reference	Compatibility
Loading sheet	9T130000	Ergo 07/08/09/SL

CADDIE



Attached to the back of the wheelchair, this caddy become quickly indispensable.



LUGGAGE RACK PACK



For a trip or a short ride, the luggage rack is a reliable and suitable accessory.

The luggage carrier bag is included in the bag.

An attachment bar is necessary on the E09L.

Name	Reference	Compatibility
Caddie	ACM00200	Ergo07/08/09
Luggage rack pack	NRM14	Ergo07/08/09
Attachment bar	9M140000	Ergo 09

ARMREST BAG



Accessory that can be easily placed on the armrest of the wheelchair, or on the handlebar of the Scooter and BiKube.



READING TRAY



Practical and easy to install on the wheelchair, the reading tray is the perfect accessory for a relaxing break on the move or at home.

Name	Reference	Compatibility
Armrest bag	ACT00300	Ergo 07/08/09/ESL
Tray	7M07000P	Ergo 07/08/09 Compact

SMARTPHONE SUPPORT

Ideal accessories for easily positioning and recharging your mobile phone on your wheelchair, and stay connected in any situation.



USB CHARGER



Name	Reference	Compatibility
Smartphone Support	7A010000	Ergo07/08/09
USB Charger	AC003000	Ergo07/08/09

LED LAMP / REFLECTOR



The LED lamp produces a long-range light to optimize night travel.

The reflector can be placed at the back.



CUP-HOLDER



Practical accessory designed to move with a cup or a glass easily.



Name	Reference	Compatibility
LED lamp	ACE00100	Ergo 07/08/09
Lamp fixation	ACE00200	Ergo 07/08/09
Reflector	ACM00300	Ergo 07/08/09
Cup-holder	ACM00100	Ergo 07/08/09

NEVER CHANGE THE HIGHLIGHTED SETTINGS

Parameterization Command E07			
Lists of configurable features		Basic settings	Observations
1	Acceleration Forward	17	Adjustable /20
2	Deceleration Forward	40	Adjustable /40
3	Lateral Acceleration	3	Adjustable /10
4	Lateral Deceleration	6	Adjustable /10
5	Minimum operating voltage	3	Adjustable
6	Maximum backward speed	5	Adjustable / 20
7	Maximum Forward speed	20	Adjustable / 20
8	Sound Level Beep	0	Binary
9	Electric lock position	1	Binary
10	Electric cut-off after 30 minutes of immobility	3	Adjustable 1=10mn
11	Left engine phase	4	Adjustable
12	Positive or negative phase	1	Binary
13	Right engine phase	4	Adjustable
14	Positive or negative phase	0	Binary
15	Basse vitesse moteur gauche	0	Adjustable
16	Low speed left motor	0	Adjustable
17	Braking delay of 0.5 seconds	20	Adjustable 1=0,5sd
18	Electric brake energy saving option	0	Binary
19	Cornering speed	10	Adjustable /20
20	Type of battery	1	Binary
21	Speed compensation	0	Adjustable

Setting an electric wheelchair is an essential phase for a correct use. It is therefore essential to be carried out by competent and trained people. Users are not allowed to change the programming of their wheelchair on their own initiative, as they do not have the programming remote control.

This setting must be made by a professional, occupational therapist, distributor or manufacturer. Failure to comply with this rule may have serious consequences for the use of the chair and lead to serious accidents for the users or those around them.

NEVER CHANGE THE HIGHLIGHTED SETTINGS

NEVER CHANGE THE HIGHLIGHTED SETTINGS

Parameterization Command E08			
Lists of configurable features		Basic settings	Observations
1	Acceleration Forward	20	Adjustable /20
2	Deceleration Forward	50	Adjustable /50
3	Lateral Acceleration	3	Adjustable /10
4	Lateral Deceleration	6	Adjustable /10
5	Minimum operating voltage	3	Adjustable
6	Maximum backward speed	5	Adjustable / 20
7	Maximum Forward speed	15	Adjustable / 20
8	Sound Level Beep	0	Binary
9	Electric lock position	1	Binary
10	Electric cut-off after 30 minutes of immobility	3	Adjustable 1=10mn
11	Left engine phase	4	Adjustable
12	Positive or negative phase	1	Binary
13	Right engine phase	4	Adjustable
14	Positive or negative phase	0	Binary
15	Basse vitesse moteur gauche	0	Adjustable
16	Low speed left motor	0	Adjustable
17	Braking delay of 0.5 seconds	0 ou 1	Adjustable 1=0,5sd
18	Electric brake energy saving option	0	Binary
19	Cornering speed	15	Adjustable /20
20	Type of battery	1	Binary
21	Speed compensation	5 à 15	Adjustable

Setting an electric wheelchair is an essential phase for a correct use. It is therefore essential to be carried out by competent and trained people. Users are not allowed to change the programming of their wheelchair on their own initiative, as they do not have the programming remote control.

This setting must be made by a professional, occupational therapist, distributor or manufacturer. Failure to comply with this rule may have serious consequences for the use of the chair and lead to serious accidents for the users or those around them.

NEVER CHANGE THE HIGHLIGHTED SETTINGS

NEVER CHANGE THE HIGHLIGHTED SETTINGS

Parameterization Command E09			
Lists of configurable features		Basic settings	Observations
1	Acceleration Forward	15	Adjustable /20
2	Deceleration Forward	17	Adjustable /20
3	Lateral Acceleration	3	Adjustable /10
4	Lateral Deceleration	6	Adjustable /10
5	Minimum operating voltage	3	Adjustable
6	Maximum backward speed	5	Adjustable / 10
7	Maximum Forward speed	20	Adjustable / 20
8	Sound Level Beep	0	Binary
9	Electric lock position	1	Binary
10	Electric cut-off after 30 minutes of immobility	3	Adjustable 1=10mn
11	Left engine phase	4	Adjustable
12	Positive or negative phase	1	Binary
13	Right engine phase	4	Adjustable
14	Positive or negative phase	0	Binary
15	Basse vitesse moteur gauche	0	Adjustable
16	Low speed left motor	0	Adjustable
17	Braking delay of 0.5 seconds	2	Adjustable 1=0,5sd
18	Electric brake energy saving option	0	Binary
19	Cornering speed	15	Adjustable /20
20	Type of battery	1	Binary
21	Speed compensation	0	Adjustable

Setting an electric wheelchair is an essential phase for a correct use. It is therefore essential to be carried out by competent and trained people. Users are not allowed to change the programming of their wheelchair on their own initiative, as they do not have the programming remote control.

This setting must be made by a professional, occupational therapist, distributor or manufacturer. Failure to comply with this rule may have serious consequences for the use of the chair and lead to serious accidents for the users or those around them.

NEVER CHANGE THE HIGHLIGHTED SETTINGS

Loan service: trying is adopting...(professionals)



In order to assist healthcare professionals in their efforts, ErgoConcept offers a solution for making equipments available.

The purpose of these loans is to allows users to try our products (wheelchairs, Scooter, BiKube, and accessories), but also to have the appropriate equipment for possible validation by occupational therapist and multidisciplinary teams.

For more information, send a message here:
contact@ergoconcept.net

Special interventions & After-sale service

Special interventions

Because each user is unique, our technicians can perform special interventions before the delivery of a wheelchair. Extend a seat depth, adjust an armrest, modify a footrest and many others... Our team studies any request and proposes personalized solutions.

After-sales service:

Ater your purchase, your reseller ensures the maintenance of your ErgoConcept mobility solution and lends you an equivalent equipment during the possible repair (within the limit of available stocks). The headquarter team is also at your disposal if you need advice based on your feedback.

More information: [Contact: sav@ergoconcept.net](mailto:sav@ergoconcept.net)



A day in Ergo 08L wheelchair...



We recently had the opportunity to share a very pleasant day with Hélène and Pierre, a funny and charming senior couple.

Faithful users of ErgoConcept solutions, they agreed to open us their home's door to film their usual activities (buying bread, using the elevator, taking a car...). A unique opportunity to better understand the daily problematics of a person with reduced mobility and their caregiver.

See the video on www.ergoconcept.net



Pierre is taking elevator with E08L



Pierre in a shop independently



Hélène easily put the E08 in her trunk



The couple goes for a ride



35 rue du petit Bois
ZAC des platières -Batiment D
69440 Mornant
FRANCE

Tel: +33 477 40 09 97
Mail: contact@ergoconcept.net

Website:
www.ergoconcept.net