BiKube®

S T A Y A C T I V E

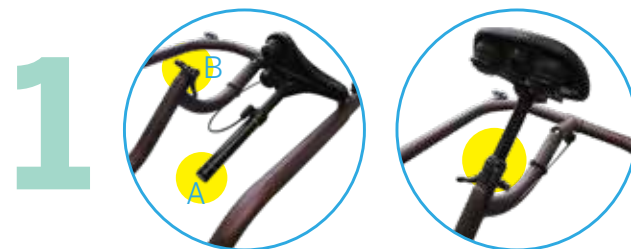
Hands-free Sit & Stand Walker
Rollator mains-libres

Before operating the BiKube®, download and read the full user guide at www.bikube.com/instructions

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
© BiKube® 2019

Insert the shaft (A) in the tube frame (B).
Make sure the cable runs along the frame and is not wrapped around the tube. Do not tighten the clamp yet.

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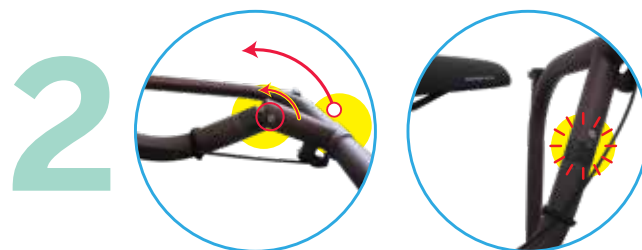Grab the top of the front frame and lift it until it is up in a straight position with the 4 wheels on the ground.

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Twist (half turn) and lock the black ring in place. You should hear a "click" when the push is engaged, and the marks on the tube and frame should face each other.

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Press the release button at the top of the frame to fully extend the seat. Using the quick release clamp, securely tighten it to the desired height for a standing position.

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Press the release button and push the seat down to its lowest position to facilitate the transition. When on the device, push the release button to shift from a sitting to a standing position.

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Insert the handlebars and secure them at the appropriate height and direction. Tip: Do not squeeze the clamp to tight and allow the handle to rotate for comfort and ease of use.

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IMPORTANT NOTICE

The ErgoConcept team would like to thank you for your trust in acquiring the BiKube® model. We have given the best of ourselves for your new BiKube® to bring you total satisfaction. Thank you for reading this entire guide BEFORE USING. ErgoConcept disclaims any liability for personal injury or property damage if the BiKube® has not been used in the strictest sense of this guide. A user must use common sense when using this product and need to consider the user's overall physical and mental health. If you are unsure if this product is appropriate for your physical or mental condition, please consult with a physician or specialist before use. Please read carefully the user guide and manual located at BiKube.com/instructions before use.

SAFETY PRECAUTIONS

This product is intended to be used solely as an indoor walking and stability aid and all other use can risk injury. BiKube® does not accept any liability for improper use or alternative use beyond what is outlined in the manual. Please adhere to the following safety warnings:

- Do not use outdoors.
- Do not use on ramps, sloped or inclined floors.
- Do not use on ramps, sloped or inclined floors or areas with uneven flooring.
- Do not use on stairs or escalators.
- Do not exceed recommended weight capacity. Maximum 250lbs/115kg
- Do not use on plush or thick carpeted areas; including throw rugs
- Do not modify or use with non-standard attachments or parts.
- Do not hang heavy items off the frame which may jeopardize its stability.
- Do not sit on the crossbar/handle area.
- Do not mount or dismount prior to engaging the wheel locks.

Important Note! This product being designed to be used indoor on a flat surface; it does not have a handbrake. Speed and stopping is controlled solely by your foot forward motion and your own stopping step.

IMPORTANT TO DO — PRIOR TO USE

- Prior to use, you should always consult your physician and/or therapist to ensure proper usage and adjustments.
- Always use caution, ensuring it is safe to operate this device.
- Ensure the device is always on a level surface before use.
- Ensure your weight is distributed evenly and directly over the seat.
- All four wheels must always be in contact with the floor during use.
- Ensure hands are clear of any pinch points when unfolding and/or adjusting the seat, height and/or handles.
- Ensure height adjustment clamps are in a locked position prior to use.
- Always ensure that the front wheels are in a locked position prior to mounting/dismounting.

- Make sure your BiKube® is properly locked before use (the 2 points of the locking ring must be aligned, and the height of your saddle must be locked with the locking ring.)

WARRANTY

Ergoconcept stands firm on the commitment to provide a quality product and exceptional service. The product has been built to precise specifications, with quality materials. It is inspected prior to shipment. Ergoconcept warrants to the original purchaser that this product will be free from defects in material and/or workman shift for the period indicated below. During the warranty period, we will correct any defects in material or workmanship at no charge. All shipping charges must be paid by the original purchaser. This product warranty is extended only to the original purchaser of this device. Proof of purchase and ownership is required to validate the warranty eligibility.

The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon transfer of ownership or use by another person. This warranty does not cover problems due to negligence, improper use of the product and/or unauthorized alterations and/or modifications. The warranty does not cover normal wear and tear and routine maintenance.

The warranty is specifically limited to the repair or replacement of the defective part and is the sole remedy of the warranty. This product is warranted for the following length of time:

- Metal frame: 2 years limited warranty
- Other parts: 1 Year limited warranty

There are no warranties which extend beyond these indicated herein and all other warranties, express or implied are hereby disclaimed. For warranty service or claims, contact the dealer from whom you purchased the BiKube®. In the event you are not able to receive satisfactory service, please contact us directly. Do not return items without our prior consent.

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